



Bovine Wormtest Kit

Use this kit for

- Wormtest;
- Wormtest with Larval culture; or
- Fluke test (composite test)



Current test prices are available from our website. Scan the QR code above or contact the laboratory. <https://nre.tas.gov.au/Documents/AHL-Price-List.pdf>

The fee includes postage, analysis and reporting of results.

Each kit provides sufficient material to test ten individual animals. Using a vinyl glove provided for your protection, simply collect dung samples from the rectum of each animal. Place the dung in one of the small jars provided. It is important to **completely fill each jar** as the exclusion of air from the sample will assist in the preservation of the sample.

Keep samples cool. Post or deliver to the Animal Health Laboratories as soon as possible. Due to current delays experienced by Australia Post, please consider using an Express Post Bag.

Worm eggs are counted in each sample and the results will usually be available within two working days of receipt of your sample at the laboratory. Results can be emailed or mailed.

Larval Culture Results

Further testing can be undertaken to identify the *types* of worms present in your cattle dung sample. Eggs can be hatched and the larvae identified. This process takes approximately 8-10 days.

More information about Wormtests and Fluke Tests can be downloaded from our website <https://nre.tas.gov.au/biosecurity-tasmania/animal-biosecurity/animal-health-laboratories/labfacts>

NOTE: An account will be forwarded to you separately.

Please do NOT send money or cheques with your sample.

In case of difficulties in conducting a Bovine Wormtest please contact your veterinarian or

Animal Health Laboratories

NRE TAS

165 Westbury Road

Prospect TAS 7250

PO Box 46

Kings Meadows

TAS 7249

T: (03) 6777 2111 E: specimenreception@nre.tas.gov.au